

NIBP adapter



SORIMEX

ul. Równinna 25, 87-100 Toruń, Poland
tel.: +48 56 657 77 20, fax: +48 56 657 77 21
e-mail: biuro@sorimex.pl ♦ www.sorimex.eu



Data ostatniej weryfikacji
Date of last verification
Letzte Verifikation
Число последней проверки
2016-01-01

1. General description

NIBP adapters are components used during non-invasive blood pressure (NIBP) measurements for connecting the cuff with the measuring device.

2. Characteristic

NIBP adapters include connectors and air tubes. Their design is adapted to the apparatus and cuff with which they are used.

3. Terms of use

Match the adapter type and size, to the cuff and apparatus with which they are to work with. Make sure that the connection made is steady and airtight and the air tubes are free from any sharp bends.

If the adapter has been damaged during work, replace it with a new one.

4. Cleaning, disinfection and sterilization

Cleaning

The outer adapter coating should be washed gently with a soft cloth soaked in warm water, with the addition of gentle cleaning agents or disinfectants if needed. 70% isopropyl alcohol can also be used. Let the adapter dry out before use. Do not use abrasive cleaners, caustic agents, solvents or any other products that could damage the adapter.

Disinfection

The solutions recommended for disinfecting adapters are 70% isopropyl alcohol or 70% ethanol. Other disinfecting agents intended for disinfecting medical devices may also be used, but only if they

do not adversely affect the adapter (before using any solutions please read the instructions for use of these

Sterilization

Sterilization with ethylene oxide is the preferred method (follow the procedure provided by the gas sterilizer supplier). After sterilization, and before use, the adapter should be safely and thoroughly aired out. Sterilization by other methods could permanently damage the adapter.

5. Storage

Adapters should be stored in their original packaging with the ambient temperature

between 0°C and 50°C and are not in use should be loosely rolled, without any sharp bends.

6. Warnings

Adapters

The device to which the adapter is connected, should meet the requirements of applicable regulations. When installing the adapter, follow the supplied instructions for use. Only qualified personnel with appropriate knowledge should connect the adapter to the device and the cuff. Disposal of the used adapter should be in accordance with applicable regulations.

7. Warranty

The
The

warranty

Symbole / Symbols / Symbole / Символы



Numer katalogowy
Catalogue number
Katalognummer
Каталожный номер
Numer partii



Batch code
Lotnummer
Номер партии



Ostrzeżenie
Caution
Warnung
Предостережение
Wytwórca



Manufacturer
Hersteller
Производитель



Data produkcji - włączona w numer partii, stanowi cyfry od 2 do 7 numeru LOT
Date of manufacture - included in the bath code, these are numbers from 2 to 7 LOT
number Herstellungsdatum - in der Lotnummer enthalten, die Zahlen 2-7 Lotnummer
Число производства - включены в номер партии, число от 2 до 7 Номер LOT
Ograniczenia temperatury



Temperature limit
Temperaturbeschränkungen
Ограничения температуры
Zapoznać się z instrukcją używania



Consult instructions for use
Sich mit der Gebrauchsanweisung vertraut machen
Ознакомиться с инструкцией обслуживания
Chronić przed światłem słonecznym



Keep away from sunlight
Vor Sonnenstrahlen schützen
Предохранять от солнечного света
Chronić przed wilgocią



Keep dry
Vor Feuchtigkeit schützen
Предохранять от сырости