SORIMEX

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EN ECG electrodes with wire

DE EKG-Elektroden

ES Electrodo de ECG con cable

SORIMEX sp. z o.o. (dawniej SORIMEX sp. z o.o. sp. k.)
Równinna 25, 87-100 Toruń, Poland
tel. +48 56 657 77 20, fax +48 56 657 77 21
office@sorimex.eu ♦ www.sorimex.eu

Rev. 2025-01-07

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PL

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EN

Instruction for disposable ECG electrodes, EK-1 type.

1. Intended use and operation of the device

ECG electrodes are intended for diagnostic tests and monitoring of the patient's heart functions.

Electrodes are placed on the skin to transmit an electrical signal from the surface of the body via ECG cables and leadwires to the ECG apparatus or cardiac monitor, which convert the signal into electrocardiogram or vector cardiogram.

2. Patient target groups

Electrodes are dedicated to use by qualified medical personnel in neonates, infants and children.

3. Contraindications

There are no known contraindications for use of the device.

4. Possible side effects

ECG electrode is a product that comes into direct contact with the patient's skin. There is a risk of irritation and allergic reactions in people with extremely sensitive skin.

The occurrence of skin reactions requires medical advice.

5. Warnings

- 5.1. Electrodes are disposable medical devices. Do not use again. Re-use may cause patients infection.
- 5.2. Use electrodes only on intact skin. Do not stick the electrode on wounded, infected or lesioned skin.
- 5.3. Do not use ECG electrodes during magnetic resonance imaging (MRI). Leaving electrodes on the skin may cause burns.
- 5.4. When performing electrosurgical procedures, place the electrodes as far away from the treatment site as possible to minimize the risk of burns to the patient's skin.

6. Cautions

- 6.1. Do not stick the electrodes on moist, lubricated skin due to the risk of the electrode coming loose.
- 6.2. Rough removal of the electrode may cause skin irritation.
- 6.3. When replacing an electrode with a new one, change the place of application of the next electrode to avoid skin irritation.
- 6.4. Use ECG electrodes only with compatible patient cables. Do not use electrodes that are wet or in a damaged packaging.
- 6.5. The wire must not be pulled out or otherwise detached from the conductive part of the electrode. Damage to the connection between the wire and the electrode may

result in permanent damage to the electrode and reduce its effectiveness.

- 6.6. Do not use electrodes which are expired.
- 6.7. Do not open the packaging when there is no necessity of using electrodes for the tests because of the risk of the gel drying out.
- 6.8. Excessive movement of the patient adversely affects the quality of the ECG signal.
- 6.9. In case the electrode comes off of the skin, replace it with the new one.
- 6.10. Do not use abrasive agents on neonates, infants and children.

7. Directions for use

- 7.1. Electrodes marked with „RTG” are intended for use in X-ray examination.
- 7.2. To connect the electrode with ECG apparatus, use ECG cables compatible with equipment used.
- 7.3. Skin at the electrode site should be clean and dry. If necessary, the skin should be washed with soap and water and dried thoroughly.
- 7.4. Optimal electrode use time: 48 hours.
- 7.5. Maximum use time of electrodes: 72 hours.
- 7.6. If a small amount of gel remains on the skin after removing the electrode, it should be wiped off with a paper towel or washed off with water.

8. Application

- 8.1. Open the pouch along the dotted line indicated on the packaging.
- 8.2. Determine the application sites on the patient's body. If necessary, prepare the patient's skin in accordance with point 7.3. Before sticking the electrode, always make sure that the application area is dry.
- 8.3. After removing the electrodes from a transparent foil, apply to previously prepared skin, taking into account the information contained in points 5, 6 and 7. Apply the electrode to the skin and gently press its edges.
- 8.4. Connect the electrode wires to the ECG cable connector or to the device that receives the ECG signals.
- 8.5. Use electrodes only in conjunction with a compatible ECG cable or device.
- 8.6. The electrode wire must be positioned to prevent accidental detachment of the electrode.

9. Electrode removal

After the examination, lift the electrode and gently, slowly detach it while holding the patient's skin with your free hand. Avoid peeling off at right angles to minimize the risk of skin irritation.

10. Storage conditions

- 10.1. Electrodes should be stored in their original packaging, at a temperature between 10°C and 40°C. Protect from direct sunlight, moisture and dirt.
- 10.2. The shelf life of the product is 2 years, it is stated on the packaging and applies to originally closed products and stored in accordance with manufacturer's instructions.

10.3. Leave unused electrodes in their original packaging and use within the next 7 days. Secure the packaging by folding its edges.

11. Explanation of symbols

Product catalogue number

Product index

Pieces per packaging

Production batch number

Medical device

Read the instruction for use

Production date

Manufacturer

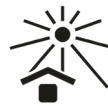
Expiration date

Caution

Keep dry

Protect against sunlight

Do not re-use

Product has an Ag/Cl sensor

Acceptable temperature limits

CE mark
(conformance to the Regulation EU 2017/745)

Product does not contain polyvinyl chloride

Product does not contain natural latex

Do not use if the package is damaged

12. Product incident reporting

Any serious incident related to the device should be reported to the manufacturer and to the authority competent for the place of residence of the user or patient.

13. Utilization

Used electrode should be disposed of in accordance with internal hygiene procedures in force at a given health care facility and with local regulations.

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