

SORIMEX

**Jednorazowe maseczki medyczne
BeSAFE Typ II**

**Disposable medical face masks
BeSAFE Type II**



 SORIMEX sp. z o.o. (dawniej SORIMEX sp. z o.o. sp. k.)
ul. Równina 25, 87-100 Toruń, Poland
tel.: +48 56 657 77 20, fax: +48 56 657 77 21
biuro@sorimex.pl • www.sorimex.eu



V, 2025-01-07

Instrukcja używania

1. Otwórz opakowanie, wyjmij i rozciągnij maseczkę, unikając dotykania jej wewnętrznej strony.
2. Trzymając za gumki na obu końcach maseczki, umieść ją na twarzy zakrywając nos, usta i podbródek. Upewnij się, że elastyczny pasek wszyty w maseczkę znajduje się na nosie.
3. Dognij pasek tak, aby dobrze przylegał do grzbietu nosa.
4. Upewnij się, że maska dobrze przylega do twarzy i podbródka, a jej dolna krawędź znajduje się pod brodą.



Instruction for use

1. Open the package, take out and stretch the mask while avoiding touching the inside of the mask.
2. Holding the ear straps on both ends of the mask, place it over your face, covering your nose, mouth and chin. Make sure that the metal elastic clip sewn into the mask is on the nose.
3. Bend the metal clip until it is snug over the bridge of your nose.
4. Make sure the mask fits snugly around the face and chin, with the bottom edge under the chin.